



Thank you for purchasing our Snappy Heat 6"x8" Reusable Heat Pack

We strive for 5-Stars. If for any reason you need help, have a question or need to contact us, please visit our website at SnappyHeat.com or email us through Amazon (seller is SDS Worldwide).

This box contains 1 Snappy Heat 6"x8" heat pack and 1 cloth holder. Please inspect your heat pack immediately for any damage caused during the shipping process.


If your pack is in a liquid form (soft), it is ready to be activated to generate heat. If it is in a solid form (hard), then you must boil it to convert it back to a liquid state prior to use. See the "Reset Instructions" below.

\$5 OFF

Get \$5 off a future Snappy Heat Purchase on Amazon.com

Request your free code today:
SnappyHeat.com/save

Activation Instructions

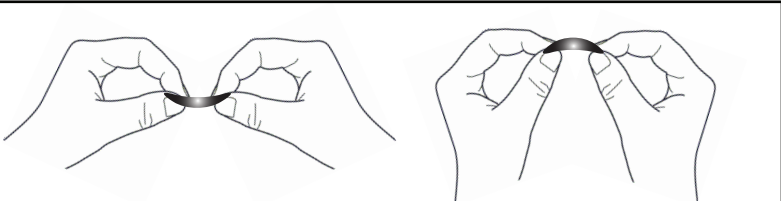

To activate your heat pack, locate the disc-like coin inside the heat pack. Hold the heat pack while placing the internal metal disc between your thumbs and index fingers and "snap" it back and forth one time. Push in the very center of the disc while holding the edges of the disc. **Do not bend the disc.** It should "pop" or as we say..."snap!" with a clicking sound. The disc must be touching the internal liquid during this process to activate. The heating process should start immediately. Knead to soften. Insert into the cloth holder and apply to enjoy.

Heat packs will last approximately 15 to 30 minutes depending on your environment. Knead again to soften and to prolong heat.

Reset Instructions

Bring a pot of water to a boil, enough to fully cover your heat packs to prevent dry contact with any hot surface. Place the heat packs into the boiling water for 15 to 18 minutes. The boil time may need to be adjusted due to number of units you are boiling at once. All the hardened crystals must be liquefied inside each heat pack. Remove with caution and let cool to room temperature. The heat packs are now ready to use again.

After removing from the water, if your heat packs automatically convert back to a crystal form, or you are unable to activate the heat packs again, this means the heat pack was not boiled long enough. Boil again for a longer period of time.

CAUTION:

DO NOT FREEZE OR MICROWAVE. DISCARD IF PUNCTURED. This product is not intended to diagnose, treat, cure or prevent any disease, ailment or injury. Surface may reach 130 degrees Fahrenheit. Discontinue use if irritation occurs. Test on a small portion of skin prior to full, direct skin contact. Keep away from small children. Do not use while under the influence of drugs or alcohol.

Online Instruction & Photos
www.SnappyHeat.com/FAQ

Ingredients: A supersaturated solution of sodium acetate (3H₂O CH₃COONa) in water.
Food-grade and non-toxic.